


Saving positions for Independent Repositioning Mode

For M3, M5, F3 & F5 models with Independent Repositioning Mode (IRM) **already enabled**.

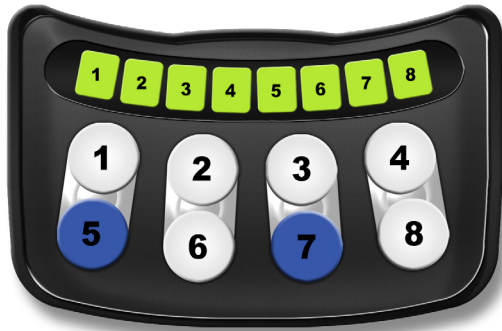

- 1 Press and hold 5 and 7 while turning the wheelchair ON.
Wait for all LED lights to turn green, then release.



- 2 LED 8 will be flashing red and green to indicate entering the programming menu.
Push button 7, LED 7 will be flashing green and LED 8 will be solid green.
You are now in IRM programming.



- 3 Utilize buttons on switch box to take client to the desired back position:
- Buttons 1 and 5 – Tilt
 - Buttons 2 and 6 – Legs
 - Buttons 3 and 7 – Recline



- 4 Press and hold button 4 to save the back position.
You will hear an audible beep that indicates the position has been successfully saved.



- 5 Use buttons on switch box to take client to desired forward or upright position:
- Buttons 1 and 5 – Tilt
 - Buttons 2 and 6 – Legs
 - Buttons 3 and 7 – Recline



- 6 Press and hold button 8 to save the forward position.
You will hear an audible beep that indicates the position has been successfully saved.
Power wheelchair off to save.