


PiezoWave2T Interface Guide


Main Menu

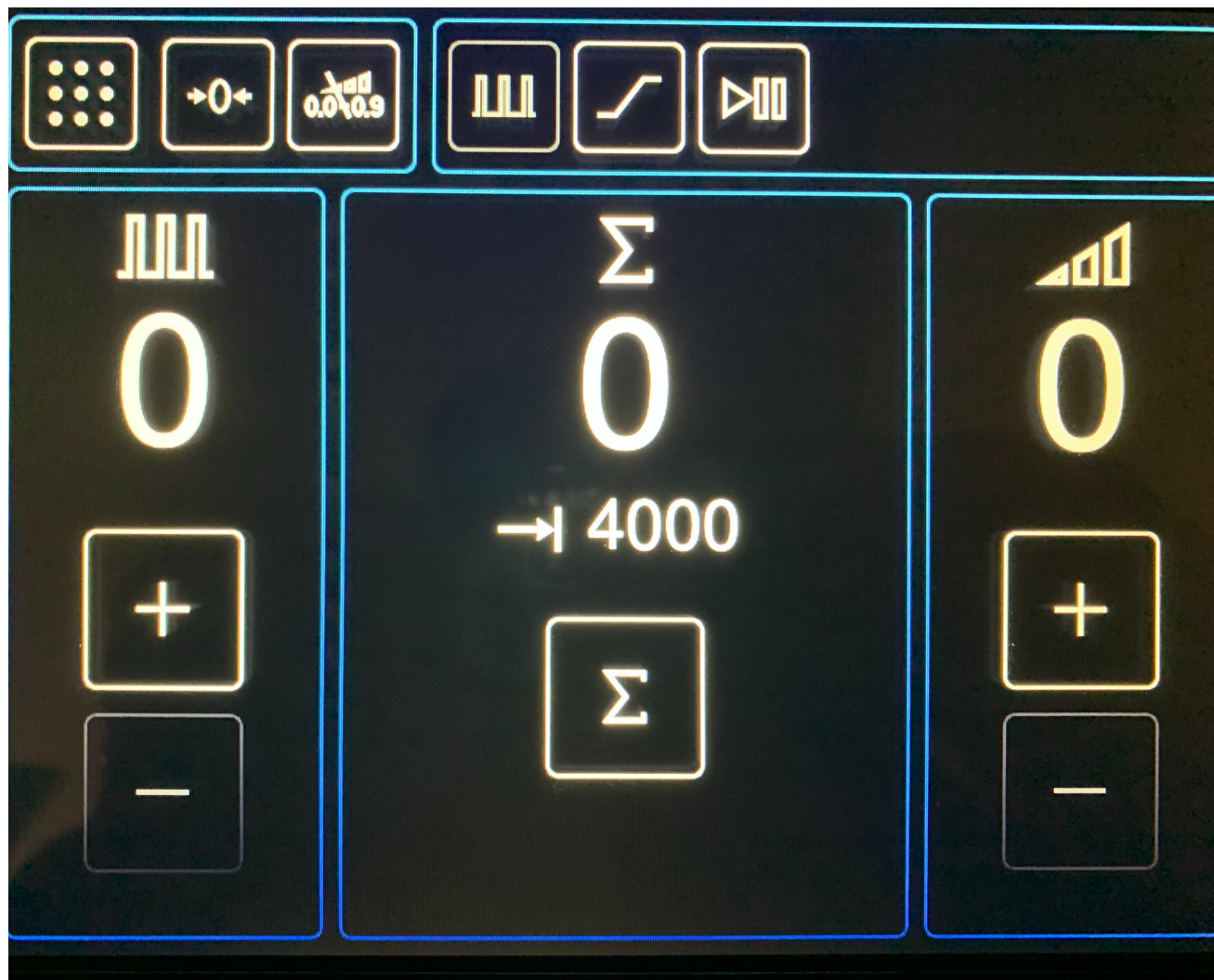

Frequency

Pulse Target Sum


Intensity

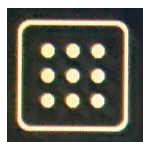


Options Menu



Reset Current
Treatment Settings

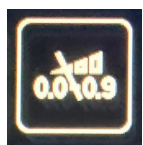


Intensity Sub-stages
(turn on/off 0.0-0.9)

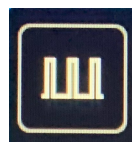


Default Treatment
Screen




RAMP Mode Screen

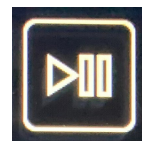

BURST Mode Screen

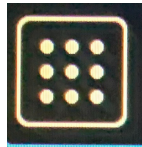


Options Menu

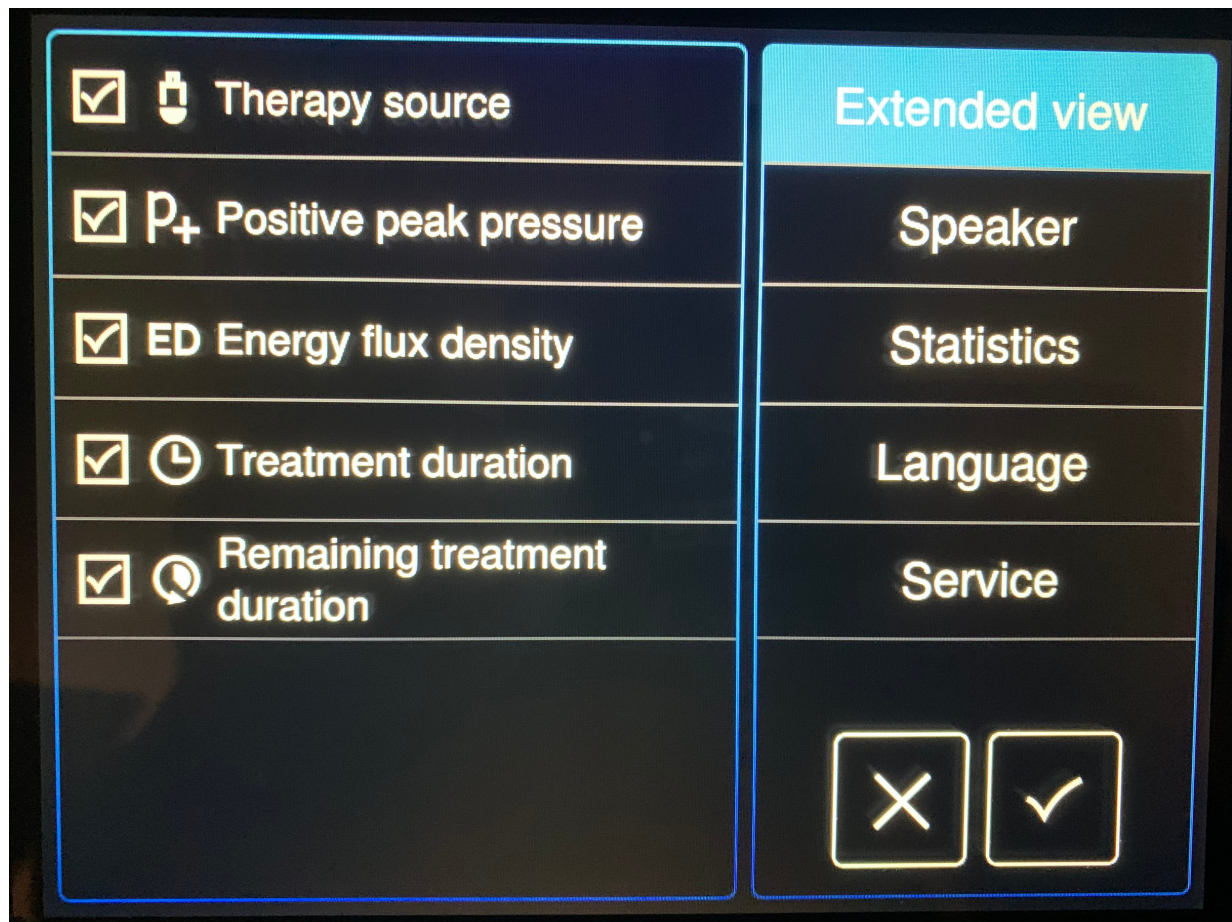

Select extended treatment parameters to show on main treatment screen.





Main screen showing selected extended parameters

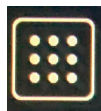


Options Menu


Statistics - Menu to show how many pulses emitted by control unit or connected therapy source.

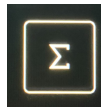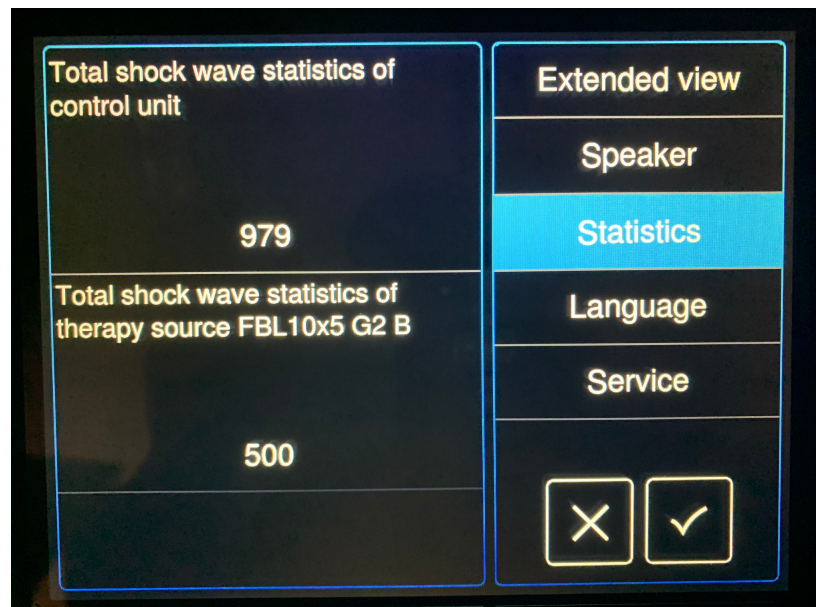


Pulse Target Sum

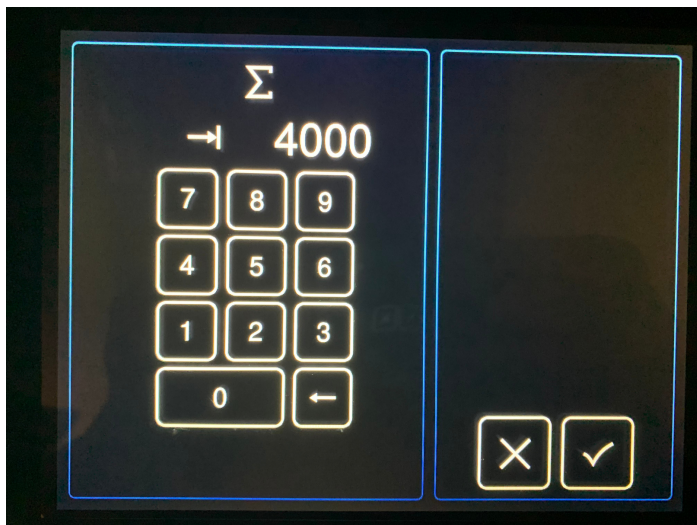


Enter number of desired pulses for treatment duration.


If at any time during treatment, you wish to update the target sum, it can be changed by pressing the  again.

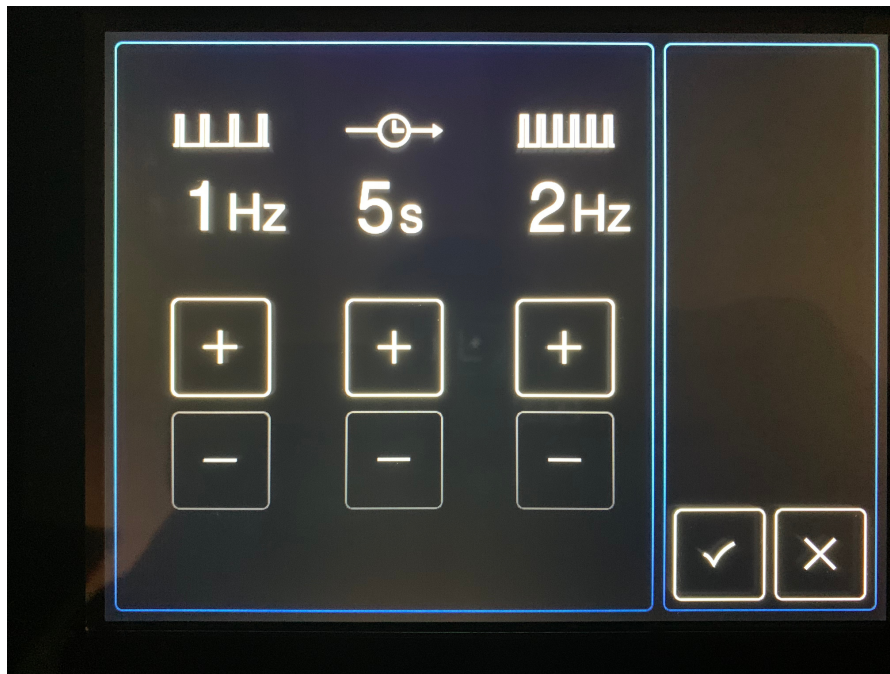


RAMP Mode Screen

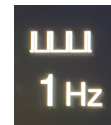



Initial Frequency

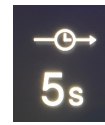


Time Frame to
Increase Frequency

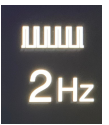


Target Frequency

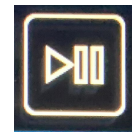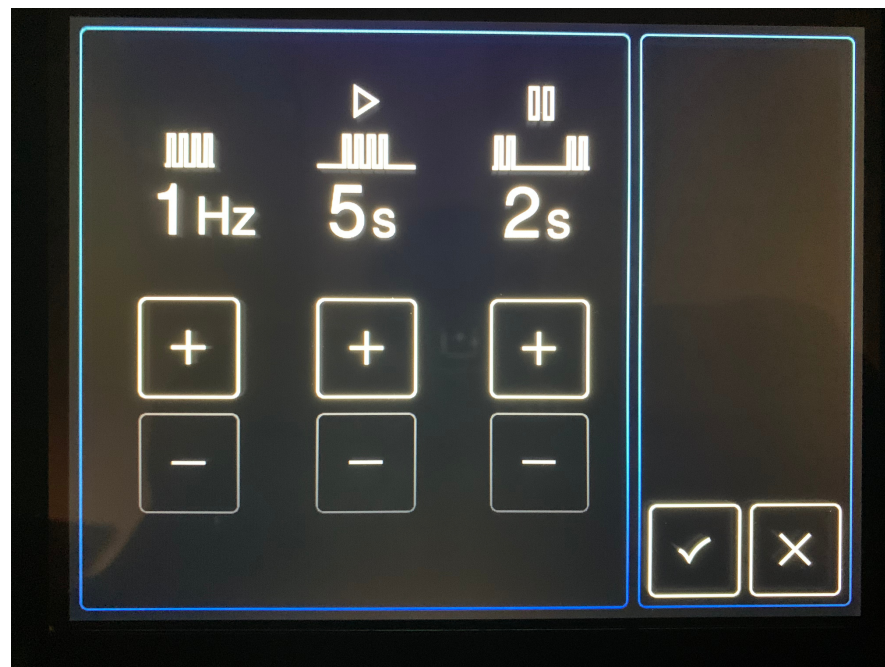


BURST Mode Screen

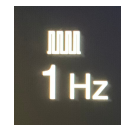

Frequency

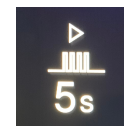

Pulse Duration

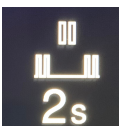

Pause Duration