




SCROLL

Left click, hold, and drag up and down

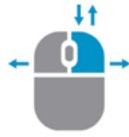


WINDOW LEVEL MEASURE / PEN

Right-click, hold, and drag up and down or side to side



ZOOM

Right and left click, hold, and drag up and down

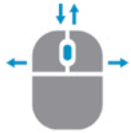

PAN

Click the mouse wheel, hold, and drag in the direction you want to pan



SPLIT SCREEN/ FULL SCREEN

Choose how to view the different series between the two



VIEW REPORT

Open the attached report (when available)



ROTATE

Rotate image clockwise



HIDE

Hide/display labels in the Viewer



INVERT

Invert grayscale of the image and background



MEASURE

Measurement on the fly



PEN

Pen tool



CINE

View sequence of image in a Cine



ADJUST IMAGE BRIGHTNESS & CONTRAST

Image display presets



FIRST ANNOTATION

Displays first annotated image in the series

Next

NEXT ANNOTATION

Displays next annotated series that is available



CHOOSE IMAGE LAYOUT

Available layouts for images in the Viewer