

Spine Bracing System PATIENT APPLICATION INSTRUCTIONS EXOS FORM™ II 626 / 627



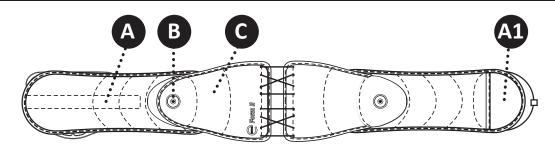

FRONT PANEL: (INCLUDED WITH MODEL 627)

Application of Brace:

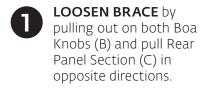



TIGHTEN BRACE by pushing in both Boa Knobs and turn knobs clockwise.

> **NOTE: Left Boa Knob** controls top half of brace. Right Boa Knob

controls bottom half.




APPLY BRACE by inserting hand into the Donning Mitt pocket (A1) and wrapping the Belt Wings (A) around the

waist.



OPTIMAL COMPRESSION: When sitting down, prior to sitting, release compression by pulling-out on Boa Knobs. Once seated, tighten compression by pushing in on Boa Knobs. When standing up, tighten compression if necessary.

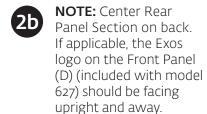





USE AND CARE (CLEANING & STORAGE

INSTRUCTIONS): Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. Do NOT iron. Do NOT expose to heat over 120°F (49°C). Do NOT tumble dry. Do NOT bleach.

MATERIAL CONTENTS: Nylon, Polyethylene, Polyester, Antimicrobial Treated Material, Polyurethane, Stainless Steel, Polycarbonate, Thermoplastic Elastomer, Polyoxymethylene, Carbon Steel, Acrylic Adhesive. FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

CONTRAINDICATIONS: Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure.



application videos, please visit DJOGlobal.com/products/exos/exos-form-ii-back-brace or call 888.405.3251

For more information including

For product assistance, please contact Product Support at 1.888.405.3251 or email product.specialist@djoglobal.com

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

Patented

Copyright @ 2017 by DJO, LLC

MKT00-4186 Rev B

Individual results may vary. Neither DJO Global, Inc. nor any of its subsidiaries dispense medical advice. The contents of this brochure do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

Next Appointment:
Date:
Time:

